



Dinterdamo

Plastic Dosers
for Veterinary Use

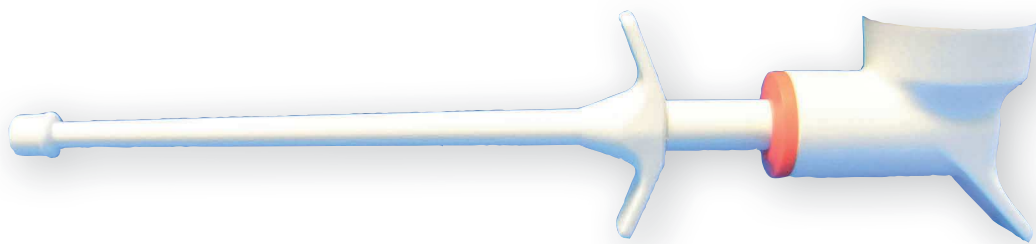

INSTRUCTIONS for USE

Screw the doser on the bottle without over-tighten to avoid deformation.

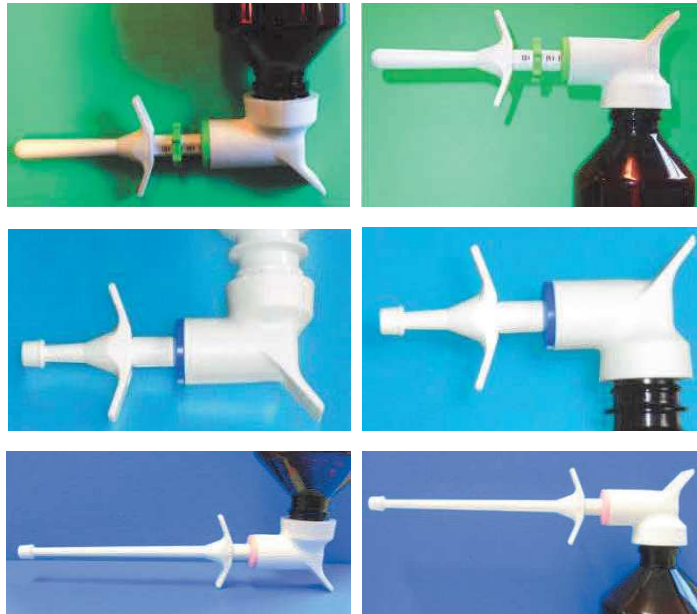

1) Viscous product:
use the upside down bottle



2) Liquid product:
use the upright bottle



Insert the tube (customer-cut)
into the dosing hole.

Cleaning and maintenance

After each use and before putting it back in the pack, completely empty the doser by residues, washing it by passing inside the water. Store in a cool, dry place and away from heat sources.

WARNING: Do not carry Doser mounted on the bottle.

Dispose of according to regulations in the country of use.

CERTIFIED QUALITY SYSTEM UNI EN ISO 9001:2015
NR. 50 100 4479 - Rev. 06